



permobil

Active Height

Clinical Resources

by Permobil Academy



Active Height

Active Height is the Permobil power seat function for vertical elevation of the seat. Common terminology of this seat function includes:

- power seat elevation
- power elevating seat
- power adjustable seat height
- high low

The seat function as an electromechanical lift that provides adjustable levels of vertical seat to floor height. The seat angles do not change with seat elevation.

Clinical benefits

Active Height for function

Wheelchair users are typically seated at a height that is functional for transfers or for optimal stability of the wheelchair base, however this height is often low relative to a person in standing. Being seated at a height that is relatively low can be challenging when a person is living or working in an environment that is not designed for a wheelchair user, as work surfaces or cupboards are positioned for a person to access in standing.

Active Height allows a person to raise their seat height, potentially bringing them to a height that allows them to access unmodified environments with increased ease, this may allow a person to participate in activities such as:

- Meal preparation – increased height allows improved vision and safer reach for cooking
- Household activity of daily living tasks, such as managing laundry or cleaning tasks
- Reaching into overhead cupboards in a kitchen or office
- Accessing office equipment – such as photocopiers, un-modified work stations



Active Height for transfers

Some wheelchair users find transferring out of a wheelchair seat challenging, this may be due to lower limb weakness limiting their ability to successfully complete a sit to stand, or due to challenges in managing a slide board transfer.

Active Height can assist with wheelchair transfers by:

- Allowing a person to raise their seat to floor height to reduce the strength demands needed to perform a sit to stand
- Optimise the influence of gravity during slide board transfers (or similar), by allowing the wheelchair seat to be slightly raised relative to the surface they are transferring to, this can be useful when a person is living or working in an unmodified environment

Active Height for social interaction

For wheelchair users that spend a significant amount of time with people who are standing, being seated at a lower height can make social interaction more challenging. Being seated at a lower height means that a wheelchair user needs to look upwards to make eye contact and/or communicate with a person in standing, this can result in a person sitting in increased cervical extension for prolonged period of time. For some users who have a significant thoracic kyphosis, achieving this upwards gaze may not be possible. *Active Height* allows a person to increase the height of their seat to allow a level gaze for eye contact and to communicate with their standing peers (Schiappa et al., 2019).



Funding

Active Height is a power seat function that requires clinical rationale to be funded. This rationale can be supported through the use of person centred goals relevant to the funding source.

Options to consider:

Transfers

What issues does the person have transferring out of their chair? How does the ability to raise the seat height assist with improving this transfer? Consider all potential transfers – bed, alternative chair, commode, etc.

Activities of daily living (ADLs)

How is the person currently completing their activity of daily living tasks? Is this consistent in different environments – e.g., work, home, family members homes, etc. How does the ability to raise the seat height assist with these tasks?

Environment

What vertical access tasks could be completed independently or with less assistance with the provision of *Active Height*? For example, would *Active Height* allow a person to access high cupboards, clothes racks, wall mounted dryers independently? Consider work, home and community environments.

Roles and responsibilities

Does the person have particular roles and responsibilities that require them to be seated at a particular height? For example, does the person have a role that requires them to make eye contact when greeting colleagues or peers as part of successfully completing their role? Are they presenting to community groups and need to be able to scan a room or be easily seen by an audience? Or are they taking care of children or other dependents and need to be able to access wardrobes and raised cupboards as part of assisting with their care?



Permobil specific


Active Height is unique in that while elevating, the seating system repositions back over the base 9 cm to maintain optimal forward stability, allowing the user to get closer to objects.

Full suspension remains available when the seat is fully elevated, so the user can keep pace with a smooth ride at a top speed of 5 km/h in full elevation.

Active Height enables a seat elevate options of up to 35 cm from the seated height.

Supporting Literature

Schiappa, V., Piriano, J., Bernhardt, L., Shea, M., Maurer, C., Rosen, L., ... & Dicianno, B. E. (2019). RESNA position on the application of seat elevating devices for power wheelchair users: literature update 2019

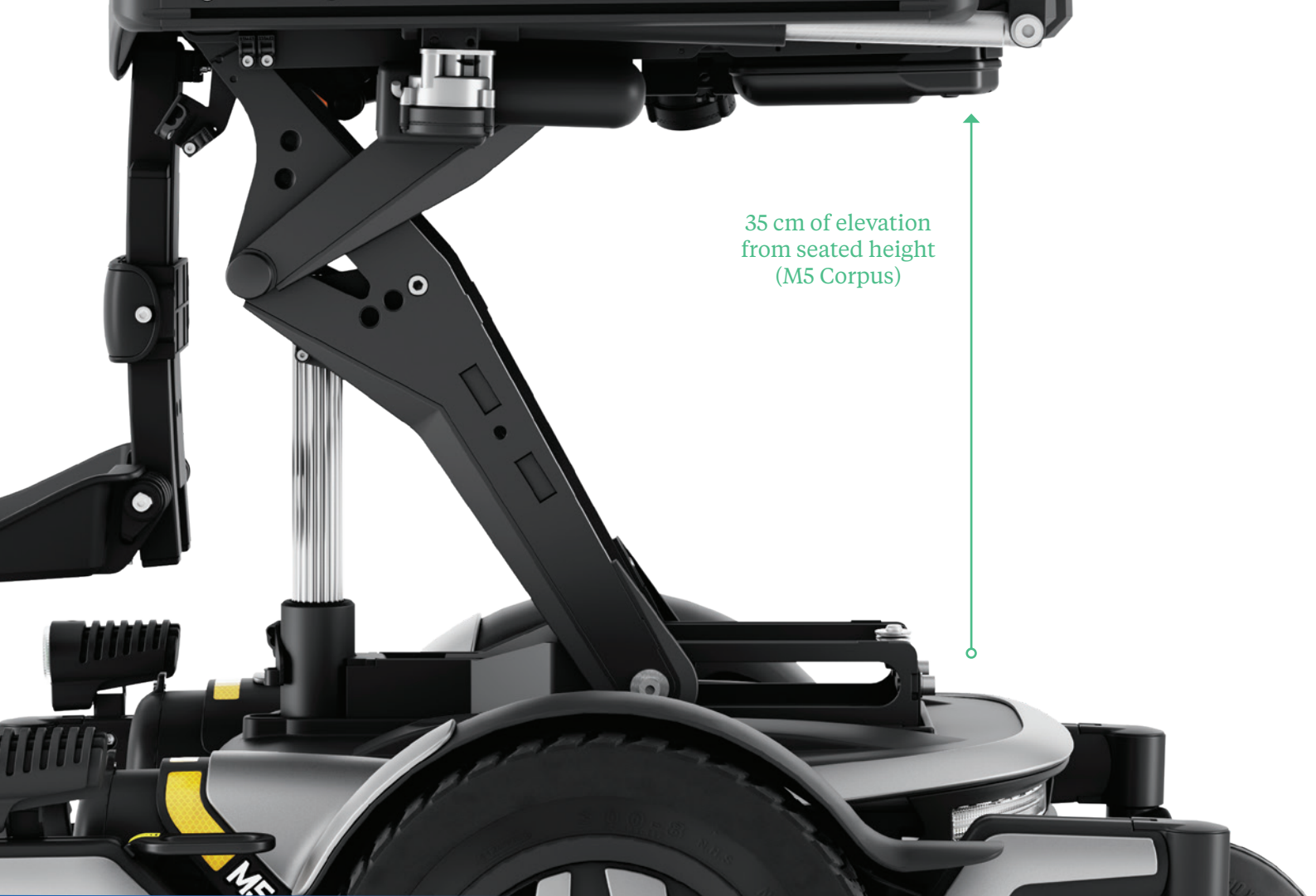

35 cm of elevation
from seated height
(M5 Corpus)

permobil

www.permobil.com